04/2023 SIG16L21IS00

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HYGIENE IS SAFETY
L'HYGIÈNE EST SYNONYME DE SÉCURITÉ
HYGIENE IST SICHERHEIT
LA HIGIENE ES SEGURIDAD

BU Medical Equipment
Sede legale ed amministrativa
Headquarters
Cefla s.c.
Via Selice Provinciale, 23/a
40026 Imola - Bo (Italy)
tel. +39 0542 653111
fax +39 0542 653344

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A product line specifically developed for dental units by those who build dental units.

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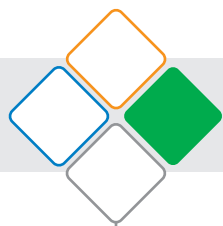
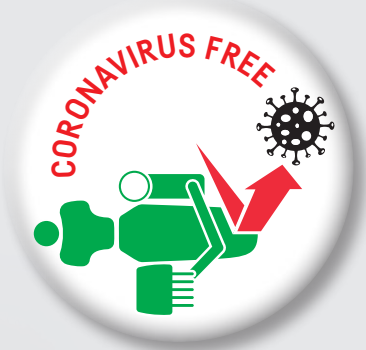
DISINFECTANT FOR DENTAL UNIT SURFACES

Tuberculicide, Bactericide, Fungicide, Virucide.
Specifically designed for the cleaning and disinfection of dental unit and patient chair surfaces.
Highly suitable for dental surgery surfaces, even delicate ones, thanks to a balanced formula with reduced alcohol content [40%].

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CONCENTRATED, HYGIENIZING FLUID FOR SURGICAL ASPIRATION SYSTEMS

High deterative and sanitising power at recommended dilution (60 ml of product in 1 litre of water).
Practical in automatic use: diluted with water at 6% in the automatic cannula wash system tank.
Effective manual use: diluted with hot water at 6% and slowly aspirated through the cannula, to ensure thorough suction tubing washing.

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DISINFECTANT FOR DENTAL UNIT WATER CONDUITS

Hydrogen peroxide and silver ion-based disinfectant, with excellent wide-spectrum biocide capacity. Specially developed for dental unit hygiene systems belonging to the brands: Anthos – Stern Weber – Castellini.

Fast acting: 10 minutes of intensive disinfection cycle contact kills 99,999% [5 Log] of bacterial load.

Control of hygiene quality of water supply: with continuous 600 ppm dilution device or manual introduction into independent feed tank.

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LUBRICANT SPRAY FOR TURBINES AND DENTAL HANDPIECES

Lubricant designed for the ultra-high performance of rotating instruments, turbines, handpieces and contra angles.
Maximum protection of the internal mechanics of instrument and maintenance of performance over time by applying daily and before each autoclave sterilisation.
Optimal, waste-free use on all instruments thanks to the set of silicon applicators supplied with the product.

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